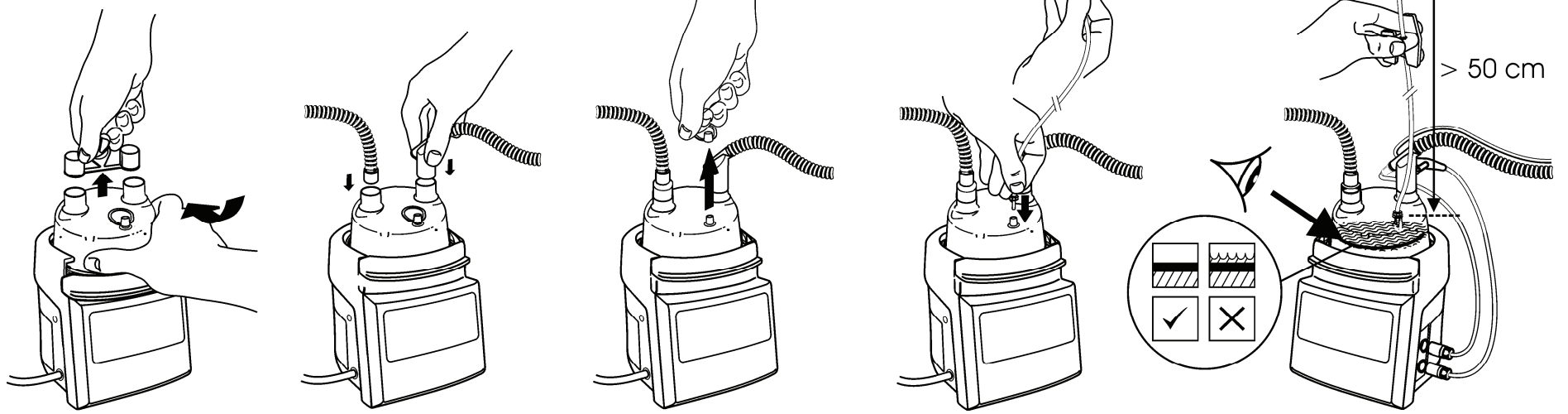
MR210, MR250

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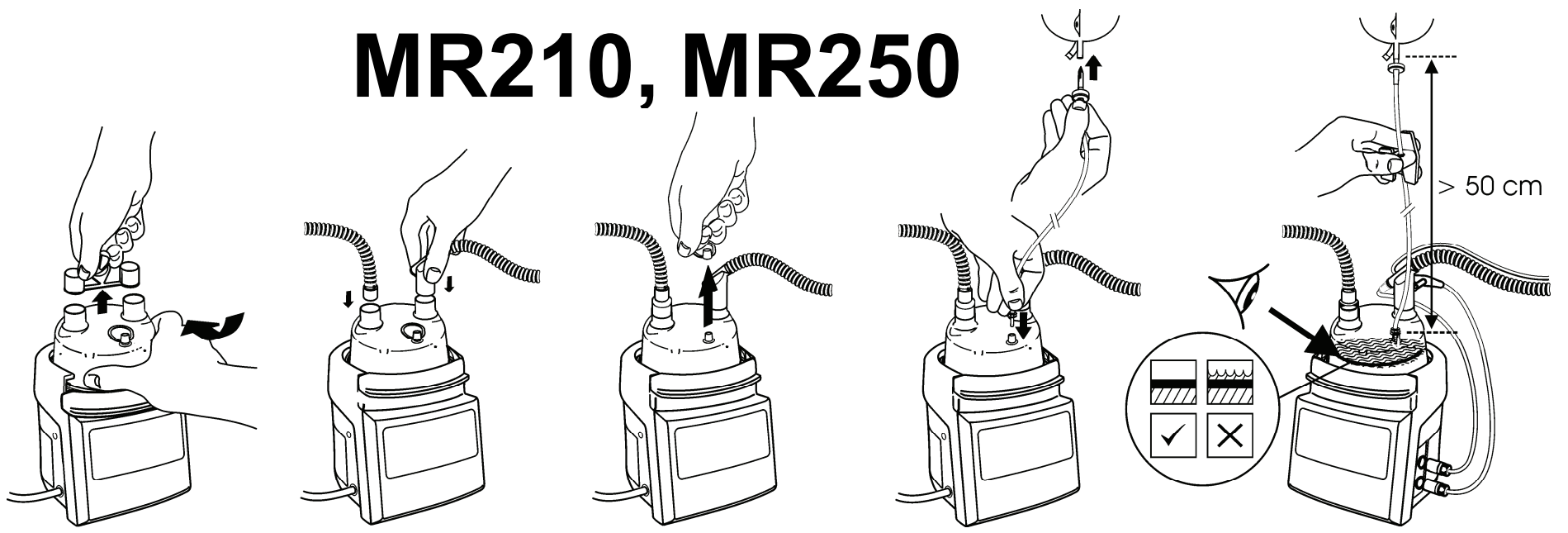
English (en)

PRODUCT SPECIFICATIONS

Gas Ports 22 mm
 MR210
 Inlet female
 Outlet male
 MR250
 Inlet male
 Outlet male

Maximum Water Capacity
 260 mL

Compliance
 0.3 to 0.5 mL/cm H₂O

Max. operating pressure
 20 kPa

Maximum peak flow
 200 L/min

Compressible Volume
 Full 220 mL
 Empty 480 mL

Water level symbol

- ✕ denotes water level too high.
- ✓ denotes maximum water level.

WARNING

- Use USP sterile water for inhalation or equivalent.
- DO NOT fill the chamber above the maximum fill level line. Liquid could enter the breathing circuit if the chamber is overfilled.
- DO NOT fill the chamber with water in excess of 37 °C.
- DO NOT use the chamber if the seals are not intact when received.
- Remove water feed inlet plug only prior to inserting water feed set. Never reinsert water feed inlet plug as this may cause leakage.
- If disconnecting the breathing circuit from the chamber the humidifier MUST be put into stand-by or turned off to prevent excessive temperatures
- Ensure the water bag is placed below the level of the humidifier after the chamber has been filled
- Ensure that the water level in the chamber is periodically monitored. Refill when necessary.
- In the event of the chamber leaking, switch the humidifier off and replace the chamber.
- Ensure the humidifier is not tilted. Tilting the humidifier may result in water entering the breathing circuit.
- DO NOT touch the heater plate or chamber base. Surfaces may exceed 85°C.
- Reuse may result in transmission of infectious substances, interruption to treatment, serious harm or death.

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