# RD020-01 Test Lung

The Fisher & Paykel Healthcare RD020 Test Lung is designed to be used for setting up or demonstrating resuscitators and ventilators at positive pressures less than 80cmH<sub>2</sub>O. The Test lung is made of silicon and is latex and DEHP free.

## Cleaning of the Test Lung

- The test lung is a consumable item. The test lung should regularly be inspected for signs of damage such as discolouration, perishing or cracking. Replace the test lung if damage is observed.
- Test Lung must be cleaned before every use or in accordance with hospital cleaning protocols.

## WARNING

- Ensure the Silicone Test Lung and Plastic connector are both removed from the T-Piece before use on a patient.

This test lung consists of two parts, the flexible silicone test lung and a rigid plastic connector. The two parts of the test lung may be disassembled for disinfection (refer Fig1). This test lung can be cleaned using disinfectants containing either peracetic acid (e.g Perasafe®) or ortho-phthaldehyde (e.g. Cidex® OPA). Other disinfection methods can cause damage to the test lung and are not recommended.

Please follow the manufacturer's instructions for storage, handling and usage of these chemicals.

\*Operating and Storage : -20° c to 50°c , up to 90% Humidity

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